

Instructions For Use



EN INSTRUCTIONS FOR USE

REF	BM33002580 – AP Instrument Protector small	STEAM
REF	BM33002585 – AP Instrument Protector medium	STEAM
REF	BM33002590 – AP Instrument Protector mini	STEAM
REF	BM33002595 – AP Instrument Protector 56mm x 86mm	STEAM

Intended use

The Instrument Protector is used as a protecting device to prevent instruments from damaging the sterile barrier system or hurting the end-user. The Instrument Protector is only intended to be used on small and sharp surgical instruments. The Instrument Protector does not serve as a Sterile Barrier System (SBS).

Intended users

- For professional use only!
- Central Sterile Services Department (CSSD) employees assigned for handling sterilization control products and ancillary products.
- Other professionals assigned for handling sterilization control products and ancillary products.

Indications and contra-indications

Use the product conform the information on the label. Do not use for any other purposes than indicated on the label.

User specifications

For using the Instrument Protector, insert the sharp edge or point of the instrument through the paper bow and make sure the device is pushed all the way down. The sharp edge or point of the instrument is inserted under the laminate. For the Instrument Protector medium, when inserting e.g. a scissor, the center flap from the top middle of the protector can be folded back/down to fixate the instrument in the Instrument Protector medium.

Handle, process and dispose in compliance with your facility/department policy and validated processes for sterilization.

Use in compliance with the indicated standards which apply for this product (see Declaration of Conformity).

Standard process values

STEAM: 134°C – 3,5 min.

Disposal information

Dispose in compliance with your facility/department policy for disposal of all sterilization packaging.

Residual risks

Risks if reused could be an incorrect representation of the tested cycle.

Do not use if any unexpected abnormalities appear in or on the product, such as, but not limited to inequalities in structure and/or color.

Avoid handling that could cause damage to the product or its properties.

Product references for use

The below shows schematic references only!



Instrument Protector small – Positioning Medical Device – Main use




Instrument Protector small and Instrument Protector medium – Positioning Medical Device – Secondary use



Instrument Protector medium – Positioning Medical Device – Main use




Instrument Protector mini – Positioning Medical Device – Main use

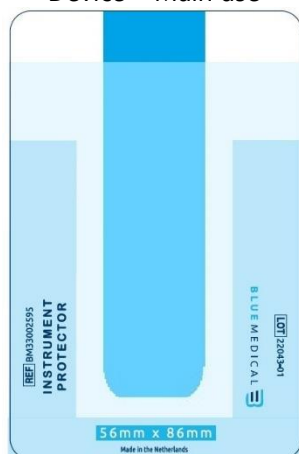

Instrument Protector 56mm x 86mm – Positioning Medical Device – Main use

Notice: any serious incident that has occurred in relation to this product should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

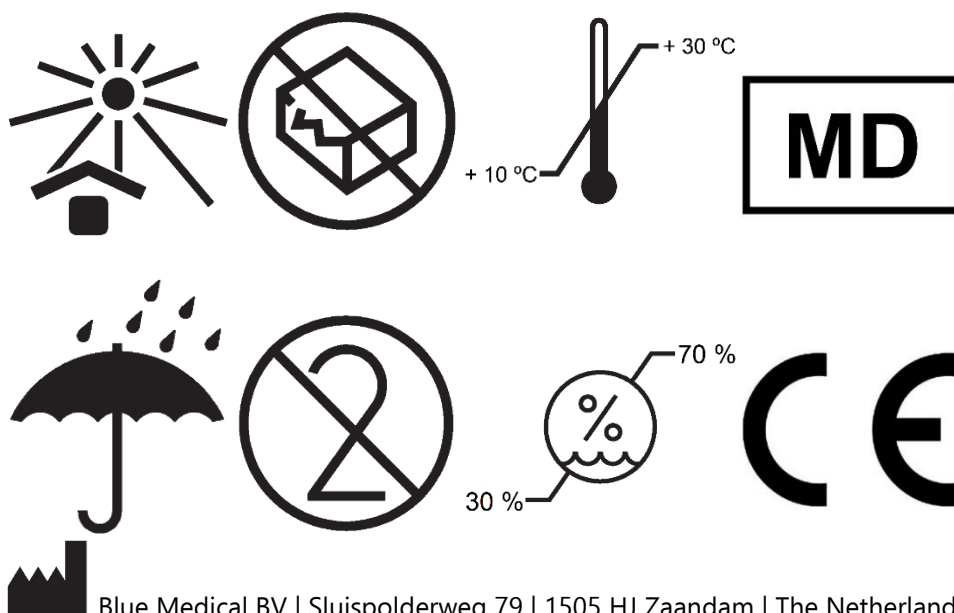

 Blue Medical BV | Sluispolderweg 79 | 1505 HJ Zaandam | The Netherlands

This Instructions for use is issued under the sole responsibility of Blue Medical B.V.. We hereby declare that the Instructions for use specified above meets the indicated standards. All supporting documentation is retained at the premises of the manufacturer.

Signature:



Mr. Alex Welleweerd
Sales Director

Place, date (dd.mm.yyyy) of issue and version:

Zaandam, 12.12.2022, v3